



AXA PARENTERALS LTD.



SURGICAL DIVISION



Sutures
Mesh
Bone Wax
Surgical Disposable

AN ISO & CE CERTIFIED SURGICAL SUTURES AND MESH MANUFACTURING COMPANY
We Manufacture complete range of Surgical Sutures, Hernia Mesh & Bonewax


Absorbable Surgical Suture




Nonabsorbable Surgical Suture




Sterile Surgical Mesh

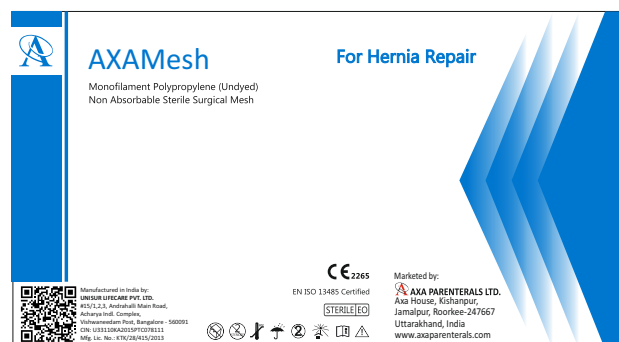

Needle Shape

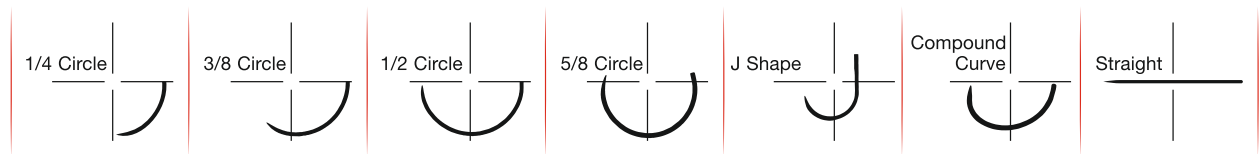


Needle Type:

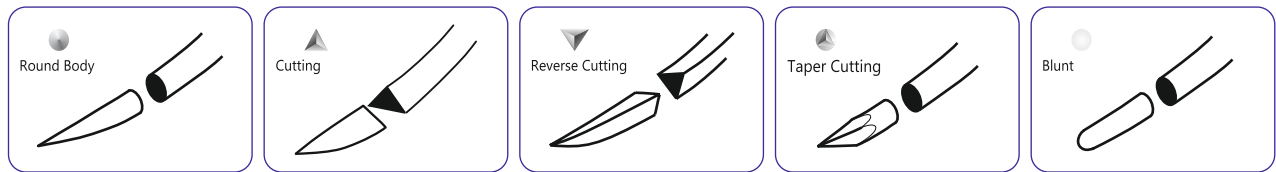

International Coding Pattern

| International Coding Pattern | | | | | | | | | | |
|------------------------------|---|--------|----------|---------------------|--------|------------------|--------|--------------------------|---------------------|--------------|
| Code 1 | Product | Code 2 | USP Size | Code 3 | Code 4 | Needle Curvature | Code 5 | Needle Type | Code 6 | |
| 1 | Braided & Coated Polyglycolic Acid Undyed | A | 3 | SUTURE LENGTH in cm | C | 3/8 Circle | Z | Round Body / Taper Point | NEEDLE LENGTH in mm | |
| 2 | Braided & Coated Polyglycolic Acid | B | 2 | | D | 1/2 Circle | Y | Cutting | | |
| 3 | Monofilament Polyamide (Nylon) | C | 1 | | A | Straight | X | Reverse Cutting | | |
| 4 | Catgut Chromic | D | 0 | | E | 5/8 Circle | K | Taper Cutting | | |
| 5 | Black Braided Silk | E | 2-0 | | | | | T | | Trocar Point |
| 6 | Braided Polyester | F | 3-0 | | | | | R | | Blunt Point |
| 7 | Catgut Plain | G | 4-0 | | | | | S | | Spatula |
| 8 | Monofilament Polypropylene | H | 5-0 | | | | | | | |
| 9 | Monofilament Polydioxanone | I | 6-0 | | | | | | | |
| 10 | Braided & Coated Polyglactin 910 | J | 7-0 | | | | | | | |
| 11 | Monofilament Polyglecaprone 25 | K | 8-0 | | | | | | | |
| 12 | Braided & Coated Polyglactin 910 Undyed | | | | | | | | | |

Example:

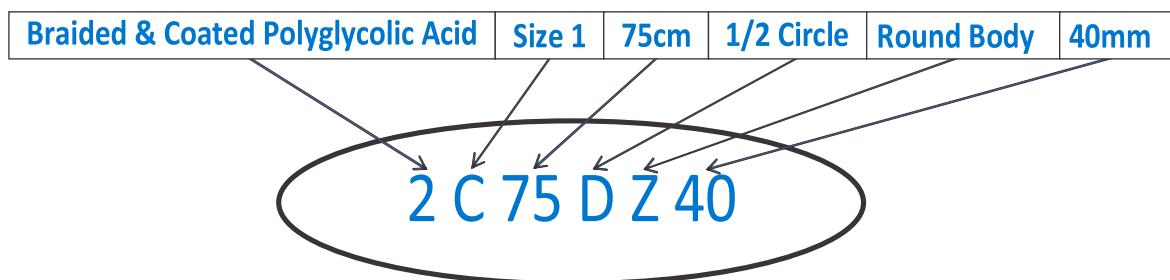



AXA PARENTERALS LTD.

AXAPga

Braided and coated Polyglycolic acid
Synthetic Absorbable Suture (U.S.P)



AXAPga

Braided and coated Polyglycolic acid Synthetic Absorbable Suture (U.S.P)



AXAPga – Braided & Coated Polyglycolic Acid Suture is an absorbable, sterile, synthetic surgical suture, composed of homopolymers of Glycolide (100%). AXAPga is also available undyed. Braided & Coated Polyglycolic Acid Suture meets all the requirements as per United States Pharmacopeia for Absorbable Surgical suture (Synthetic)

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications of Polyglycolic Acid Suture

Polyglycolic Acid suture is indicated for use in general soft tissue approximation and/or ligation, including use in ophthalmic procedures, but not for use in cardiovascular or neurological tissues.

- Smooth Surface provides no tissue trauma
- Secure Knot holding capacity
- Best suture for critical wound support
- Provides Best tissue support even after smaller diameter
- Hydrolytic absorption ensures no tissue reaction

Additional Information:

| | | |
|-------------------------|---|---|
| Characteristics | : | Synthetic Absorbable Surgical Suture |
| Type | : | Braided |
| Material | : | 100% Polyglycolic Acid synthetic absorbable braided surgical suture |
| Coating | : | Polycaprolactone & calcium stearate |
| Color | : | Violet |
| Support | : | 28 to 30 days |
| Absorption | : | Absorption takes place through hydrolysis completely in 70 to 90 days. |
| Strength | : | 75% tensile strength is retained for 14 days and 50% till 21 days. |
| USP Range | : | 6/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Packaging | : | In aluminium foil packages with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXAPga

Braided and coated Polyglycolic acid
Synthetic Absorbable Suture (U.S.P)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|------------------|----------|--------|--|
| | USP SIZE | LENGTH | |
| S 2613 | 2-0 | 150 cm | Without Needle |
| S 2613MDT | 2-0 | 180 cm | Without Needle |
| S 2614 | 0 | 150 cm | Without Needle |
| S 2614MDT | 0 | 180 cm | Without Needle |
| S 2615 | 1 | 150 cm | Without Needle |
| S 2615MDT | 1 | 180 cm | Without Needle |
| S 2777MDT/18 0 F | 2-0 | 180 cm | Without Needle |
| S 2777MDT F | 2-0 | 50 cm | Without Needle |
| AS 2228 | 2 | 90 cm | 1/2 Circle Round Bodied 45mm |
| AS 2228VS | 2 | 110 cm | 1/2 Circle Round Bodied 45mm |
| AS 2303 | 5-0 | 45 cm | 1/2 Circle Round Bodied 16mm |
| AS 2304 | 4-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 2305 | 4-0 | 70 cm | 1/2 Circle Round Bodied 16mm |
| AS 2317 | 2-0 | 90 cm | 1/2 Circle Round Bodied 30mm |
| AS 2317DA | 2-0 | 140 cm | 1/2 CRB 30mm & 1/2 CTC 30mm Double Armed |
| AS 2319 | 1 | 90 cm | 1/2 Circle Taper Cutting 36mm OB Needle |
| AS 2338 | 0 | 90 cm | 1/2 Circle Round Bodied 30mm |
| AS 2341 | 2-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 2342 | 0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 2343 | 1 | 20 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 2344 | 0 | 20 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 2345 | 2-0 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346 | 0 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346A | 0 | 45 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346AA | 0 | 20 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346DA | 0 | 140 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2346DA/180 | 0 | 180 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2346VS | 0 | 110 cm | 1/2 Circle Round Bodied 40mm |
| AS 2347 | 1 | 90 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347A | 1 | 45 cm | 1/2 Circle Round Bodied 40mm Heavy |

AXAPga

Braided and coated Polyglycolic acid
Synthetic Absorbable Suture (U.S.P)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 2347AA | 1 | 20 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347VS | 1 | 110 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347DA | 1 | 140 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2347DA/160 | 1 | 160 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2347DA/180 | 1 | 180 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2350 | 1 | 90 cm | 1/2 Circle Reverse Cutting 40mm |
| AS 2359 | 1 | 70 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2361 | 0 | 45 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2362 | 1 | 45 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2363 | 1 | 70 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2364 | 1 | 90 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2364VS | 1 | 110 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2382 | 2-0 | 90 cm | 1/2 Circle Reverse Cutting 40mm |
| AS 2401 | 3-0 | 45 cm | 3/8 Circle Cutting 22mm |
| AS 2402 | 3-0 | 70 cm | 3/8 Circle Cutting 19mm |
| AS 2407 (UASdyed) | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 2421 | 1 | 90 cm | 1/2 Circle Reverse Cutting 40mm OR needle |
| AS 2421DA- OR | 1 | 140 cm | 1/2 CRB 40mm & 1/2 CRC 40mm OR needle |
| AS 2422 | 0 | 90 cm | 1/2 Circle Reverse Cutting 40mm |
| AS 2437 | 3-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 2438 | 1 | 90 cm | 1/2 Circle Reverse Cutting 50mm Heavy |
| AS 2442 | 5-0 | 45 cm | 3/8 Circle Cutting 16mm |
| AS 2448 | 1 | 100 cm | 1/2 Circle Round Bodied 50mm Heavy |
| AS 2465 | 4-0 | 45 cm | 5/8 Circle Cutting 16mm |
| AS 2471 | 3-0 | 45 cm | 3/8 Circle Cutting 16mm |
| AS 2472 | 3-0 | 90 cm | 1/2 Circle Cutting 22mm |
| AS 2473 | 2-0 | 70 cm | 5/8 Circle Round Bodied 27mm |
| AS 2493 | 5-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 2494 | 4-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 2495 | 3-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |

AXAPga

Braided and coated Polyglycolic acid
Synthetic Absorbable Suture (U.S.P)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|------------------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 2511 | 2-0 | 90 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2511A | 2-0 | 45 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2511AA | 2-0 | 20 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2512 | 0 | 90 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2512A | 0 | 160 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2513 | 1 | 230 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2513A | 1 | 300 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2513AA | 1 | 20 cm | 1/2 Circle Taper Cutting 30mm |
| AS 2515 | 3-0 | 45 cm | 1/2 Circle Taper Cutting 17mm |
| AS 2516 | 3-0 | 90 cm | 1/2 Circle Taper Cutting 36mm |
| AS 2517 | 2-0 | 90 cm | 1/2 Circle Taper Cutting 40mm |
| AS 2518 | 0 | 90 cm | 1/2 Circle Taper Cutting 40mm |
| AS 2518VS | 0 | 110 cm | 1/2 Circle Taper Cutting 40mm |
| AS 2534 | 0 | 90 cm | 1/2 Circle Reverse Cutting 36mm OR Needle |
| AS 2546 | 0 | 70 cm | 1/2 Circle Round Bodied 40mm |
| AS 2825A | 0 | 35 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2826 | 1 | 70 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2826A | 1 | 35 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2944 | 4-0 | 70 cm | 3/8 Circle Reverse Cutting 16mm |
| AS 945 (UASdyed) | 2-0 | 90 cm | 1/2 Circle Round Bodied 36mm |
| AS 946 (UASdyed) | 0 | 90 cm | 1/2 Circle Round Bodied 36mm |
| AS 2347DA/180AS | 1 | 180 cm | 1/2 Circle Round Bodied 50mm Heavy Double Armed |
| AS 2437DA | 3-0 | 70 cm | 3/8 Circle Round Bodied 22mm Double Armed |
| AS 2306 | 4-0 | 70 cm | 3/8 Circle Cutting 16mm |
| AS 2304VS | 4-0 | 20 cm | 1/2 Circle Round Bodied 20mm |
| AS 2511A-AS | 2-0 | 45 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 2517AS | 2-0 | 90 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 2518AS | 0 | 90 cm | 1/2 Circle Taper Cutting 45mm |
| AS 2478 | 2 | 90 cm | 1/2 Circle Reverse Cutting 40mm OR NEEDLE |
| AS 2347DA/180 RB | 1 | 180 cm | 1/2 Circle Round Bodied 40mm Double Armed |

AXAPga

Braided and coated Polyglycolic acid
Synthetic Absorbable Suture (U.S.P)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|------------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 2382A | 2-0 | 45 cm | 1/2 Circle Reverse Cutting 40mm |
| AS 2421A | 1 | 45 cm | 1/2 Circle Reverse Cutting 40mm OR NEEDLE |
| AS 2473LDL | 2-0 | 100 cm | 5/8 Circle Round Bodied 27mm |
| AS 2831RB | 6-0 | 45 cm | 3/8 Circle Round Bodied 13mm |



AXA PARENTERALS LTD.

AXACRYL

Braided & Coated Polyglactin 910
Synthetic Absorbable Surgical Suture U.S.P.



AXACRYL

Braided & Coated Polyglactin 910
Synthetic Absorbable Surgical Suture U.S.P.



AXACRYL – Braided & Coated Polyglactin 910 Suture is a synthetic absorbable sterile surgical suture composed of a copolymer made from 90% glycolide and 10% L-lactide

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing

Indications:

AXACRYL Polyglactin 910 synthetic absorbable suture is indicated only for use in superficial soft tissue approximation of the skin and mucosa. The suture holds its tensile strength for approximately two to three weeks in tissue and is completely absorbed by hydrolysis within 56 to 70 days.

- Secure Knot holding capacity
- Smooth surface provides easy tissue passage
- Better Tensile Strength provides wound support upto 28 to 32 days

Best Used For:

- Uterus approximation
- Soft tissue approximation

Additional Information:

| | | |
|-------------------------|---|---|
| Suture Characteristics | : | Synthetic Absorbable Surgical Suture |
| Suture Type | : | Braided |
| Suture Material | : | Polyglycolide (90%)- co-lactide(10%) synthetic absorbable braided surgical suture |
| Suture Coating | : | Poly (glycolide-ce-L-lactide) & calcium stearate |
| Suture Color | : | Violet |
| Suture Wound Support | : | 28 to 32 days |
| Suture Absorption | : | Absorption takes place through hydrolysis completely in 56 to 70 days. |
| Suture Tensile Strength | : | 75% tensile strength is retained for 14 days and 50% till 21 days. |
| Suture USP Range | : | 6/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Suture Packaging | : | In aluminium foil packages with or without needle |
| Suture Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXACRYL

Braided & Coated Polyglactin 910
Synthetic Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|----------------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 2317 | 2-0 | 90 cm | 1/2 Circle Round Bodied 30mm |
| AS 2317DA | 2-0 | 140 cm | 1/2 CRB 30mm & 1/2 CTC 30mm Double Armed |
| AS 2345 | 2-0 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346 | 0 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 2346DA | 0 | 140 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2346VS | 0 | 110 cm | 1/2 Circle Round Bodied 40mm |
| AS 2228 | 2 | 90 cm | 1/2 Circle Round Bodied 45mm |
| AS 2347 | 1 | 90 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2359 | 1 | 70 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347A | 1 | 45 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347VS | 1 | 110 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2347VS+ | 1 | 120 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2437 | 3-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 2442 | 5-0 | 45 cm | 3/8 Circle Cutting 16mm |
| AS 2443 | 4-0 | 70 cm | 3/8 Circle Cutting 16mm |
| AS 2421 | 1 | 90 cm | 1/2 Circle Reverse Cutting 40mm OR Needle |
| AS 2421A | 1 | 45 cm | 1/2 Circle Reverse Cutting 40mm OR Needle |
| AS 2534 | 0 | 90 cm | 1/2 Circle Reverse Cutting 36mm OR Needle |
| AS 2518 | 0 | 90 cm | 1/2 Circle Taper Cutting 40mm |
| AS 2382 | 2-0 | 90 cm | 1/2 Circle Reverse Cutting 40mm |
| AS 2448 | 1 | 100 cm | 1/2 Circle Round Bodied 50mm |
| AS 2448LN | 1 | 100 cm | 1/2 Circle Round Bodied 55mm |
| AS 2448DA/180 | 1 | 180 cm | 1/2 CRB 50mm & 1/2 CTC 50mm Double Armed |
| AS 2160DA/160 | 1 | 160 cm | 1/2 CTC 40mm & 3/8 CRB 50mm Double Armed |
| AS 2304 | 4-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 2340 | 3-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 2340A | 3-0 | 45 cm | 1/2 Circle Round Bodied 30mm |
| AS 2347DA | 1 | 140 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2347DA/180 | 1 | 180 cm | 1/2 CRB 40mm & 1/2 CTC 40mm Double Armed |
| AS 2347DA/180N | 1 | 180 cm | 1/2 CRB 50mm & 1/2 CTC 40mm Double Armed |

AXACRYL

Braided & Coated Polyglactin 910
Synthetic Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 2347VSS | 1 | 125 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2350 RB | 1 | 120 cm | 1/2 Circle Round Bodied 36mm |
| AS 2360 | 1 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 2363 | 1 | 70 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 2404 | 2-0 | 90 cm | 3/8 Circle Cutting 26mm |
| AS 2438 N | 1 | 90 cm | 1/2 Circle Reverse Cutting 45mm |
| AS 2471 | 3-0 | 45 cm | 3/8 Circle Cutting 16mm |
| AS 2516 | 3-0 | 90 cm | 1/2 Circle Taper Cutting 36mm |
| AS 2519 | 1 | 90 cm | 1/2 Circle Taper Cutting 40mm |
| AS 947 (Undyed) | 1 | 90 cm | 1/2 Circle Taper Point 36mm |
| AS 2401 | 3-0 | 45 cm | 3/8 Circle Cutting 22mm |
| AS 2473N | 2-0 | 90 cm | 5/8 Circle Round Bodied 27mm |
| AS 2517OB | 0 | 90 cm | 1/2 Circle Taper Cutting 36mm |
| AS 2518DA/180 | 0 | 180 cm | 1/2 Circle Taper Cutting 40mm |
| AS 2826 | 1 | 70 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2944 | 4-0 | 70 cm | 3/8 Circle Reverse Cutting 16mm |
| AS 2347DA/180- NN | 1 | 180 cm | 1/2 CRB 40mm Heavy & 1/2 CTC 40mm Heavy 40mm Double Armed |
| AS 2438 | 1 | 90 cm | 1/2 Circle Reverse Cutting 50mm Heavy |
| AS 2437A | 3-0 | 70 cm | 1/2 Circle Round Bodied 16mm |
| AS 945 | 2-0 | 90 cm | 1/2 Circle Round Bodied 36mm |
| AS 2494 | 4-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 2472 | 3-0 | 90 cm | 1/2 Circle Cutting 22mm |
| AS 946 (Undyed) | 0 | 90 cm | 1/2 Circle Round Bodied 36mm |
| AS 2304SP | 4-0 | 75 cm | Straight Taper Point 19mm |
| AS 2338 | 0 | 90 cm | 1/2 Circle Round Bodied 30mm |
| AS 2347N | 1 | 115 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2825 | 0 | 45 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2826A | 1 | 35 cm | 1/2 Circle Reverse Cutting 23mm |
| AS 2831CZ | 6-0 | 75 cm | 3/8 Circle Round Bodied 13mm |
| AS 2831RB | 6-0 | 45 cm | 3/8 Circle Round Bodied 13mm |

AXACRYL

Braided & Coated Polyglactin 910
Synthetic Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------|----------|--------|--|
| | USP SIZE | LENGTH | |
| 10I45CX11DA | 6-0 | 45 cm | 3/8 Circle Reverse Cutting DA 11mm |
| 10H75CX13 | 5-0 | 75 cm | 3/8 Circle Reverse Cutting 13mm |
| 10G75CZ26 | 4-0 | 75 cm | 3/8 Circle Round Bodied 26mm |
| 10G75CY31 | 4-0 | 75 cm | 3/8 Circle Cutting 31 mm |
| 10G75CY16 | 4-0 | 75 cm | 3/8 Circle Cutting 16 mm |
| 10G75CX16 | 4-0 | 75 cm | 3/8 Circle Reverse Cutting 16 mm |
| 10G70CZ16 | 4-0 | 70 cm | 3/8 Circle Round Bodied 16 mm |
| 10F75DZ26 | 3-0 | 75 cm | 1/2 Circle Round Bodied 26 mm |
| 10F75DZ22 | 3-0 | 75 cm | 1/2 Circle Round Bodied 22 mm |
| 10F75DZ19 | 3-0 | 75 cm | 1/2 Circle Round Bodied 19 mm |
| 10F75CZ26 | 3-0 | 75 cm | 3/8 Circle Round Bodied 26 mm |
| 10F75CY26 | 3-0 | 75 cm | 3/8 Circle Cutting 26 mm |
| 10F70CY16 | 3-0 | 70 cm | 3/8 Circle Cutting 16 mm |
| 10F70CX16 | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 16 mm |
| 10E75DZ31 | 2-0 | 75 cm | 1/2 Circle Round Bodied 31 mm |
| 10D90DZ40 | 0 | 90 cm | 1/2 Circle Round Bodied 40 mm |
| 10D75DZ31 | 0 | 75 cm | 1/2 Circle Round Bodied 31 mm |
| 10C90DZ40 | 1 | 90 cm | 1/2 Circle Round Bodied 40 mm |
| 10C75DZ40 | 1 | 75 cm | 1/2 Circle Round Bodied 40 mm |
| 10E75DZ26 | 2-0 | 75 cm | 1/2 Circle Round Bodied 26 mm |
| 10D75DZ40H | 0 | 75 cm | 1/2 Circle Taper Point 40mm Heavy Needle |
| 10C75DZ45H | 1 | 75 cm | 1/2 Circle Taper Point 45mm Heavy Needle |
| 10B75DZ48H | 2 | 75 cm | 1/2 Circle Taper Point 48mm Heavy Needle |
| 10E75DZ26H | 2-0 | 75 cm | 1/2 Circle Taper Point 26mm Heavy Needle |
| 10C65CR65 | 1 | 65 cm | 3/8 Circle Blunt Point 65mm |
| 10F45CX26 | 3-0 | 45 cm | 3/8 Circle Reverse Cutting 26mm |
| 10F45CZ26H | 3-0 | 45 cm | 3/8 Circle Taper Point 26mm Heavy Needle |
| 10E75CX26 | 2-0 | 75 cm | 3/8 Circle Reverse Cutting 36mm |



AXA PARENTERALS LTD.

AXACRYL FAST

Braided & Coated Polyglactin 910
(Fast absorbable)

Synthetic Absorbable Suture U.S.P.



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

AXACRYL FAST

Braided & Coated Polyglactin 910 (Fast absorbable)
Synthetic Absorbable Suture U.S.P.



AXACRYL FAST– Braided & Coated Polyglactin 910 Suture is a synthetic absorbable sterile surgical suture composed of a low molecular weight copolymer made from 90% glycolide and 10% L-lactide.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing

Indications:

AXACRYL FAST Polyglactin 910 synthetic absorbable suture is indicated only for use in superficial soft tissue approximation of the skin and mucosa, where only short term wound support (7-10 days) is required.

- Secure Knot holding capacity
- Smooth surface provides easy tissue passage

Best Used For:

- Uterus approximation
- Soft tissue approximation

Additional Information:

| | | |
|-------------------------|---|---|
| Suture Characteristics | : | Synthetic Absorbable Surgical Suture |
| Suture Type | : | Braided |
| Suture Material | : | Polyglycolide (90%)- co-lactide(10%) synthetic absorbable braided surgical suture |
| Suture Coating | : | Poly (glycolide-co-L-lactide) & calcium stearate |
| Suture Color | : | Undyed |
| Suture Wound Support | : | 7 to 10 days |
| Suture Absorption | : | Absorption takes place through hydrolysis completely in 40 to 45 days. |
| Suture Tensile Strength | : | 75% tensile strength is retained for 3 days and 50% till 5 days |
| Suture USP Range | : | 4/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 3 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Suture Packaging | : | In aluminium foil packages with or without needle |
| Suture Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXACRYL FAST

Braided & Coated Polyglactin 910 (Fast absorbable)
Synthetic Absorbable Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|---------------|----------|--------|--|
| | USP SIZE | LENGTH | |
| AS 2739 F | 3-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 2762DY F | 2-0 | 90 cm | 1/2 Circle Cutting 30mm Double Armed |
| AS 2762F VS | 2-0 | 110 cm | 1/2 Circle Round Bodied 35mm |
| AS 2777 F | 2-0 | 140 cm | 1/2 CRB 36mm & 1/2 CRC 36mm Double Armed |
| AS 2718 F | 4-0 | 45 cm | 3/8 Circle Cutting 16mm |
| AS 2516RB F | 3-0 | 90 cm | 1/2 Circle Round Bodied 35mm |
| AS 2718RB F | 4-0 | 45 cm | 3/8 Circle Round Bodied 16mm CIRKUM |
| AS 2760 F | 2-0 | 75 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 2732 F | 3-0 | 75 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 2732 AF | 3-0 | 45 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 2762 FS | 2-0 | 100 cm | 1/2 Circle Taper Cutting 35mm EPICIO |
| AS 2761 F | 2-0 | 90 cm | 1/2 Circle Round Bodied 38mm |
| AS 2763 FS | 0 | 100 cm | 1/2 Circle Taper Cutting 35mm |
| AS 2762 F | 2-0 | 90 cm | 1/2 Circle Taper Cutting 35mm |
| AS 2763 F | 0 | 90 cm | 1/2 Circle Taper Cutting 35mm |
| AS 2763 FDA | 0 | 90 cm | 1/2 Circle Taper Cutting 35mm Double Armed |
| AS 2735A FAST | 3-0 | 45 cm | 1/2 Circle Cutting 22mm |



AXA PARENTERALS LTD.

AXAMono

Monofilament Polyglecaprone 25
Synthetic Absorbable Suture U.S.P.



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

AXAMono

Monofilament Polyglecaprone 25 Synthetic Absorbable Suture U.S.P.



AXAMono - Polyglecaprone 25 suture is a monofilament synthetic absorbable surgical suture prepared from a copolymer of glycolide and epsilon-caprolactone. Monofilament Polyglecaprone 25 suture copolymer has been found to be non antigenic, non pyrogenic and elicits only a slight tissue reaction during absorption.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications:

Polyglecaprone 25 sutures are indicated for use in general soft tissue approximation and/or ligation, but not for use in cardiovascular or neurological tissues, microsurgery or ophthalmic surgery.

Ultimate Suture For:

- Female's skin approximation
- Subcutaneous approximation

Best Used By:

- Plastic Surgeons,
- Gynaecologists,
- Tricologists,
- General Surgeon,
- Orthopaedics

Additional Information:

| | | |
|-------------------------|---|--|
| Characteristics | : | Synthetic Absorbable Surgical Suture |
| Type | : | Monofilament |
| Material | : | 75%poly (glycolide) – 25% co-caprolactone synthetic absorbable surgical suture |
| Coating | : | – |
| Color | : | Undyed / Violet |
| Wound Support | : | 21 to 28 days |
| Absorption | : | Absorption takes place through hydrolysis completely in 90 to 120 days. |
| Tensile Strength | : | 50% tensile strength is retained for 14 days. |
| USP Range | : | 4/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 3 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Packaging | : | In aluminium foil packages with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXAMono

Monofilament Polyglecaprone 25
Synthetic Absorbable Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|----------|----------|--------|--------------------|
| | USP SIZE | LENGTH | |

| | | | |
|-------------------------------------|-----|--------|------------------------------------|
| Polyglecaprone 25 Undyed Codes | | | |
| S 1614 | 0 | 150 cm | Without Needle |
| N 1205 | 4-0 | 70 cm | 3/8 Circle Reverse Cutting 16mm |
| N 1326 | 3-0 | 70 cm | 3/8 Circle Cutting 25mm PRIME |
| N 1326A | 3-0 | 45 cm | 3/8 Circle Cutting 25mm PRIME |
| N 3326 | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 26mm |
| N 1641 | 2-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| N 1642 | 1-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| N 1651/35 | 2-0 | 35 cm | Straight Cutting 60mm |
| N 3203 | 5-0 | 45 cm | 3/8 Circle Reverse Cutting 13mm |
| N 3204 | 5-0 | 45 cm | 3/8 Circle Reverse Cutting 16mm |
| Polyglecaprone 25 Dyed Violet Codes | | | |
| N 4228M | 2 | 90 cm | 1/2 Circle Round Bodied 45mm |
| N 4241M | 2-0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| N 4242M | 0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| N 4246M | 0 | 76 cm | 1/2 Circle Round Bodied 40mm |
| N 4259M | 1 | 76 cm | 1/2 Circle Round Bodied 40mm |
| S 2214M | 0 | 152 cm | Without Needle |
| S 2215M | 1 | 152 cm | Without Needle |
| N 2214M | 0 | 152 cm | 1/2 Circle Round Bodied 30mm |
| N 2215M | 1 | 152 cm | 1/2 Circle Round Bodied 40mm |
| N 4226M | 1 | 76 cm | 3/8 Circle Round Bodied 45mm |
| N 4273M | 1 | 76 cm | 1/2 Circle Reverse Cutting 50mm |
| N 4245M | 2-0 | 76 cm | 1/2 Circle Round Bodied 40mm |
| N 4248M | 1 | 100 cm | 1/2 Circle Round Bodied 50mm Heavy |
| N 4255M | 2-0 | 76 cm | 1/2 Circle Cutting 30mm Heavy |
| N 4258M | 0 | 76 cm | 1/2 Circle Round Bodied 40mm Heavy |
| N 1326DY | 3-0 | 70 cm | 3/8 Circle Cutting 25mm PRIME |
| N 1326A DY | 3-0 | 45 cm | 3/8 Circle Cutting 25mm PRIME |
| N 1326A DY | 3-0 | 45 cm | 3/8 Circle Cutting 25mm PRIME |
| 11H90DZ14 | 5-0 | 90 cm | 1/2 Circle Round Body 14 mm |



AXA PARENTERALS LTD.

AXAXanone

Monofilament Polydioxanone
Synthetic Absorbable Suture U.S.P.



AXAXanone

Monofilament Polydioxanone Synthetic Absorbable Suture U.S.P.



AXAXanone – Monofilament Polydioxanone Suture is prepared from the polyester, poly (p-dioxanone).

Indications:

Two important characteristics describe the in vivo performance of absorbable sutures first tensile strength retention and second the absorption rate (loss of mass). AXAXanone synthetic absorbable suture has been formulated to minimize the variability of these characteristics and to provide wound support through an extended healing period.

- Where Delayed Wound Support required
- Surface as smooth as tissues; provides no tissue drag during suturing

Ultimate Suture For:

- Geriatric Surgeries
- Surgeries on Obese Patient
- Surgeries on Diabetic Patient
- Oncology Patients
- Surgeries on Patients have delayed healing strength

Best Used By:

- Joint muscles approximation
- Hard tissue approximation
- Gluteus Maximus approximation
- AXAXanone – Only hope of wound support during muscle movement

Additional Information:

| | | |
|-------------------------|---|---|
| Suture Characteristics | : | Synthetic Absorbable Surgical Suture |
| Type | : | Monofilament |
| Material | : | Polyester, poly-p-dioxanone |
| Coating | : | — |
| Color | : | Violet |
| Wound Support | : | 35 to 45 days |
| Absorption | : | Absorption takes place through hydrolysis completely in 180 to 210 days. |
| Tensile Strength | : | 50% tensile strength is retained for 30 days. |
| USP Range | : | 6/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 3 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Packaging | : | In aluminium foil packages with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXAXanone

Monofilament Polydioxanone
Synthetic Absorbable Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-----------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 9133 | 2-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 9132 | 3-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 9210 | 1-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| AS 9233 | 1-0 | 70 cm | 1/2 Circle Round Bodied 40mm |
| AS 9234 | 1 | 90 cm | 1/2 Circle Round Bodied 40mm |
| AS 9248 | 1 | 90 cm | 1/2 Circle Round Bodied 45mm |
| AS 9237 | 3-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 9304 | 4-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 9371 | 1-0 | 90 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 9352 | 1 | 90 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 9262 | 1 | 150 cm | 1/2 Circle Round Bodied 50mm Heavy LOOP |
| AS 9261 | 0 | 150 cm | 1/2 Circle Round Bodied 50mm Heavy LOOP |
| AS 9221 | 1 | 90 cm | 1/2 Circle Reverse Cutting 40mm OR Needle |
| AS 9201H | 5-0 | 70 cm | 1/2 Circle Round Bodied 13mm Double Armed |
| AS 9235SA | 1 | 110 cm | 1/2 Circle Taper Cutting 30mm |



AXA PARENTERALS LTD.

AXAChromic

Catgut Chromic
Natural Absorbable Suture (U.S.P.)



AXAPlain

Catgut Plain
Natural Absorbable Suture (U.S.P.)



AXAChromic

Catgut Chromic Natural Absorbable Suture (U.S.P.)



AXAChromic - Catgut Chromic Suture is an absorbable sterile surgical gut suture composed of purified connective tissue (mostly collagen) derived from the sub mucosal fibrous layer of healthy sheep intestines. Catgut Chromic is processed to provide greater resistance to absorption, chromic catgut treated with chromic salts to delay the absorption.

Indications of Chromic catgut Suture

Chromic Suture is indicated for use in general soft tissue approximation and/or ligation, including use in ophthalmic procedures, but not for use in cardiovascular and neurological tissues.

Ultimate suture for:

- Uterus Closure
- Episiotomy Repair
- Soft Tissues approximation
- Fast healing tissues/ Muscles

Additional Information:

| | | |
|-------------------------|---|---|
| Suture Characteristics | : | Absorbable Surgical Suture |
| Type | : | Twisted |
| Material | : | Catgut Chromic |
| Color | : | Brown |
| Absorption | : | Absorption takes place due to enzymatic reaction in 90 to 120 days. |
| Tensile Strength | : | Tensile strength is retained for 14 to 21 days. |
| USP Range | : | 5/0 – 3 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction |
| Packaging | : | In aluminium foil packages with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point. |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXAChromic

Catgut Chromic
Natural Absorbable Suture (U.S.P.)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-----------|----------|--------|------------------------------------|
| | USP SIZE | LENGTH | |
| AS 2214 | 0 | 152 cm | 1/2 Circle Round Bodied 30mm |
| AS 2215 | 1 | 152 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 2216 | 2 | 152 cm | 1/2 Circle Round Bodied 45mm MAYO |
| AS 4113 | 2-0 | 76 cm | 3/8 Circle Round Bodied 32mm |
| AS 4114 | 0 | 76 cm | 3/8 Circle Round Bodied 32mm |
| AS 4123 | 2-0 | 76 cm | 1/2 Circle Round Bodied 26mm |
| AS 4124 | 0 | 76 cm | 1/2 Circle Round Bodied 26mm |
| AS 4200 | 2-0 | 76 cm | Straight Round Bodied 40mm |
| AS 4201 | 3-0 | 76 cm | 3/8 Circle Cutting 22mm |
| AS 4204 | 2-0 | 76 cm | Straight Round Bodied 45mm |
| AS 4205 | 0 | 76 cm | Straight Round Bodied 45mm |
| AS 4208 | 2-0 | 76 cm | Straight Round Bodied 50mm |
| AS 4213 | 0 | 76 cm | Straight Round Bodied 60mm |
| AS 4216 | 2-0 | 76 cm | 3/8 Circle Round Bodied 30mm |
| AS 4217 | 0 | 76 cm | 3/8 Circle Round Bodied 30mm |
| AS 4220 | 2-0 | 76 cm | 3/8 Circle Round Bodied 40mm |
| AS 4221 | 0 | 76 cm | 3/8 Circle Round Bodied 40mm |
| AS 4225 | 0 | 76 cm | 3/8 Circle Round Bodied 45mm |
| AS 4226 | 1 | 76 cm | 3/8 Circle Round Bodied 45mm |
| AS 4227 | 1 | 100 cm | 1/2 Circle Round Bodied 45mm MAYO |
| AS 4228 | 2 | 100 cm | 1/2 Circle Round Bodied 45mm MAYO |
| AS 4229 | 0 | 100 cm | 1/2 Circle Round Bodied 45mm MAYO |
| AS 4229VS | 0 | 110 cm | 1/2 Circle Round Bodied 45mm MAYO |
| AS 4232 | 2-0 | 76 cm | 3/8 Circle Round Bodied 65mm |
| AS 4233 | 0 | 76 cm | 3/8 Circle Round Bodied 65mm |
| AS 4237 | 3-0 | 76 cm | 1/2 Circle Round Bodied 20mm |
| AS 4237AS | 3-0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| AS 4239 | 1 | 100 cm | 1/2 Circle Trocar Point 40mm Heavy |
| AS 4239A | 1 | 76 cm | 1/2 Circle Trocar Point 40mm Heavy |
| AS 4241 | 2-0 | 76 cm | 1/2 Circle Round Bodied 30mm |

AXAChromic

Catgut Chromic
Natural Absorbable Suture (U.S.P.)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-----------|----------|--------|---|
| | USP SIZE | LENGTH | |
| AS 4242 | 0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| AS 4244 | 2 | 100 cm | 1/2 Circle Trocar Point 40mm Heavy |
| AS 4245 | 2-0 | 76 cm | 1/2 Circle Round Bodied 40mm |
| AS 4246 | 0 | 76 cm | 1/2 Circle Round Bodied 40mm |
| AS 4248 | 1 | 100 cm | 1/2 Circle Round Bodied 50mm Heavy |
| AS 4249 | 2-0 | 76 cm | 1/2 Circle Round Bodied 45mm |
| AS 4250 | 0 | 76 cm | 1/2 Circle Round Bodied 45mm |
| AS 4255 | 2-0 | 76 cm | 1/2 Circle Cutting 30mm Heavy |
| AS 4258 | 0 | 76 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 4259 | 1 | 76 cm | 1/2 Circle Round Bodied 40mm Heavy |
| AS 4262 | 1 | 76 cm | 3/8 Circle Cutting 60mm |
| AS 4264 | 0 | 76 cm | 5/8 Circle Round Bodied 35mm |
| AS 4267 | 1 | 76 cm | 3/8 Circle Round Bodied Blunt 63mm |
| AS 4268 | 5-0 | 76 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 4272 | 3-0 | 76 cm | 1/2 Circle Reverse Cutting 12mm |
| AS 4273 | 1 | 76 cm | 1/2 Circle Reverse Cutting 50mm |
| AS 4274 | 3-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 4275 | 1 | 152 cm | 1/2 CRB 45mm & 1/2 CRC 45mm Double Armed |
| AS 4276 | 0 | 152 cm | 1/2 CRB 45mm & 1/2 CRC 45mm Double Armed |
| AS 4277 | 2-0 | 152 cm | 1/2 CRB 36mm & 1/2 CRC 36mm Double Armed |
| AS 4278 | 2 | 100 cm | 1/2 Circle Round Bodied 65mm |
| AS 4279 | 0 | 100 cm | 1/2 Circle Taper Cutting H 40mm |
| AS 4280 | 4-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 4281 | 5-0 | 76 cm | 3/8 Circle Round Bodied 16mm |
| AS 4281CY | 5-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 4295 | 2-0 | 76 cm | 1/2 Circle Round Bodied 38mm |
| AS 4296 | 0 | 76 cm | 1/2 Circle Round Bodied 38mm |
| AS 4297 | 1 | 100 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 4297A | 1 | 76 cm | 1/2 Circle Taper Cutting 40mm Heavy |
| AS 4826 | 1 | 76 cm | 3/8 Circle Round Bodied 45mm Double Armed |

AXAChromic

Catgut Chromic
Natural Absorbable Suture (U.S.P.)



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|----------|----------|-----------|-------------------------------|
| | USP SIZE | LENGTH | |
| N 5048 | 4-0 | 76 cm | 3/8 Circle Round Bodied 16 mm |
| N 4204 | 2-0 | 76 cm | Straight Round Bodied 45mm |
| S 2212 | 3-0 | 152 cm | Without Needle |
| S 2213 | 2-0 | 152 cm | Without Needle |
| S 2214 | 0 | 152 cm | Without Needle |
| S 2215 | 1 | 152 cm | Without Needle |
| S 2216 | 2 | 152 cm | Without Needle |
| S 2217 | 3 | 152 cm | Without Needle |
| S 2255 | 1 | 2 X 76 cm | Without Needle |
| 4E75DZ26 | 2-0 | 75 cm | 1/2 Circle Round Bodied 26 mm |
| 4E76DZ26 | 2-0 | 76 cm | 1/2 Circle Round Bodied 26 mm |
| 4D76DZ40 | 0 | 76 cm | 1/2 Circle Round Bodied 40 mm |
| 4C76DZ40 | 1 | 76 cm | 1/2 Circle Round Bodied 40 mm |
| 4D75DZ40 | 0 | 75 cm | 1/2 Circle Round Bodied 40 mm |
| 4C75DZ40 | 1 | 75 cm | 1/2 Circle Round Bodied 40 mm |
| 7F76DZ26 | 3-0 | 76 cm | 1/2 Circle Round Bodied 26 mm |
| 7F76DZ25 | 3-0 | 76 cm | 1/2 Circle Round Bodied 25 mm |
| 7E76DZ31 | 2-0 | 76 cm | 1/2 Circle Round Bodied 31 mm |

AXAPlain

Catgut Plain

Absorbable Surgical Suture U.S.P.



AXAPlain - Catgut Plain Suture is a fast absorbing surgical gut suture made from a strand of collagenous material prepared from the submucosal layers of the small intestine of healthy sheep. Fast absorbing surgical gut sutures are sterile and elicit only a slight to minimal tissue reaction during absorption.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications of Plain Catgut Suture

Plain Catgut Suture is fast absorbing and intended for dermal (skin) suturing only. They should be utilized only for external knot tying procedures.

Best Used By:

- General Surgeons
- Gynaecologists
- Orthopaedics

Ultimate Suture for:

- Uterus Closure
- Episiotomy Repair
- Soft Tissues approximation
- Fast healing tissues/ Muscles


Additional Information:

| | | |
|-------------------------|---|---|
| Suture Characteristics | : | Absorbable Surgical Suture |
| Type | : | Twisted |
| Material | : | Catgut Plain |
| Color | : | Beige |
| Absorption | : | Plain Catgut Suture Absorption takes place due to enzymatic reaction in 60 to 70 days |
| Strength | : | Tensile strength is retained for 7 – 10 days. |
| USP Range | : | 4/0 – 3 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction. |
| Packaging | : | In aluminum foil packages with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXAPlain

Catgut Plain

Absorbable Surgical Suture U.S.P.

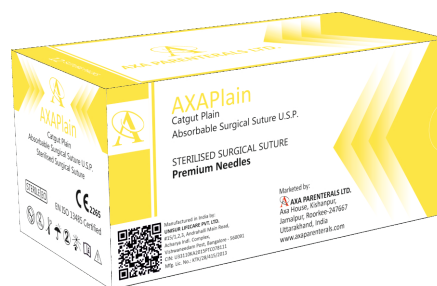

| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------|----------|-----------|--------------------------------------|
| | USP SIZE | LENGTH | |
| AS 4238 | 3-0 | 76 cm | 1/2 Circle Round Bodied 25mm |
| AS 4269 | 5-0 | 76 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 4270 | 4-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 4271 | 3-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 4288 | 3-0 | 76 cm | Straight Cutting 60mm |
| AS 5007 | 3-0 | 76 cm | Straight Round Bodied 50mm |
| AS 5010 | 3-0 | 76 cm | 3/8 Circle Cutting 22mm |
| AS 5020 | 3-0 | 76 cm | 3/8 Circle Cutting 26mm |
| AS 5033 | 2-0 | 76 cm | 3/8 Circle Round Bodied 30mm |
| AS 2003 LDL | 2-0 | 152 cm | 1/2 Circle Cutting 30mm Double Armed |
| S 2002 | 3-0 | 152 cm | Without Needle |
| S 2003 | 2-0 | 152 cm | Without Needle |
| S 2004 | 0 | 152 cm | Without Needle |
| S 2005 | 1 | 152 cm | Without Needle |
| S 2006 | 2 | 152 cm | Without Needle |
| S 2242 | 3-0 | 2 x 76 cm | Without Needle |



AXA PARENTERALS LTD.

ULTRALon

Monofilament Polyamide
Non Absorbable Surgical Suture U.S.P.



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

ULTRALon

Monofilament Polyamide Non Absorbable Surgical Suture U.S.P.



ULTRALon – Monofilament Polyamide (Nylon) is a non absorbable sterile surgical suture, composed of the long-chain aliphatic homo polymers Nylon 6 and Nylon 6,6.

ULTRALon Monofilament Polyamide Sutures are dyed black or blue to enhance visibility in tissue.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications:

Monofilament Polyamide Suture is indicated for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological procedures.

- The best material for skin approximation
- Being a monofilament with smooth surface never support bacterial growth followed by capillary action
- Uniform elasticity supports small movements results cosmetic effect
- Ultra smooth surface results negligible pain

Best Used By:

- General Surgeons,
- Gynaecologists,
- Orthopaedics

Ultimate Suture For:

- Skin Closure
- Hard Tissues approximation
- Casualty stitching

Additional Information:

| | | |
|-------------------------|---|--|
| Suture Characteristics | : | Non Absorbable Surgical Suture |
| Type | : | Monofilament |
| Material | : | Polyamide (Nylon) |
| Color | : | Black |
| Absorption | : | Non Absorbable |
| USP Range | : | 6/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction. |
| Packaging | : | In food grade folder packed in a medical grade pouch, with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

ULTRALon

Monofilament Polyamide
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-----------|----------|--------|---------------------------------------|
| | USP SIZE | LENGTH | |
| AS 3303 | 2-0 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 3304 | 0 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 3304L | 0 | 100 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 3305 | 1 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 3306 | 2 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 3315 | 5-0 | 35 cm | 3/8 Circle Cutting 16mm |
| AS 3315RB | 5-0 | 75 cm | 3/8 Circle Round Bodied 16mm |
| AS 3316 | 5-0 | 75 cm | 3/8 Circle Round Bodied 12mm |
| AS 3317 | 5-0 | 70 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 3318 | 4-0 | 70 cm | 3/8 Circle Cutting 16mm |
| AS 3319RB | 4-0 | 75 cm | 1/2 Circle Round Bodied 19mm |
| AS 3320AS | 6-0 | 75 cm | 3/8 Circle Reverse Cutting 13mm |
| AS 3320RB | 6-0 | 75 cm | 3/8 Circle Round Bodied 13mm |
| AS 3321 | 3-0 | 70 cm | 3/8 Circle Cutting 16mm |
| AS 3326AS | 4-0 | 38 cm | 3/8 Circle Cutting 10mm |
| AS 3328 | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 3328T | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 3328A | 3-0 | 35 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 3328AT | 3-0 | 35 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 3331 | 6-0 | 70 cm | 3/8 Circle Cutting 26mm |
| AS 3336 | 2-0 | 70 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 3336T | 2-0 | 70 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 3336A | 2-0 | 35 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 3336AT | 2-0 | 35 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 3336DK | 2-0 | 110 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 3338 | 1 | 100 cm | 1/2 Circle Reverse Cutting 45mm Heavy |
| AS 3338A | 1 | 45 cm | 1/2 Circle Reverse Cutting 45mm Heavy |
| AS 3340 | 0 (Loop) | 150 cm | 1/2 Circle Round Bodied 40mm |
| AS 3341 | 2-0 | 75 cm | 1/2 Circle Round Bodied 30mm |
| AS 3346 | 0 | 100 cm | 1/2 Circle Round Bodied 40mm Heavy |

ULTRALon

Monofilament Polyamide

Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------|----------|-----------|-------------------------------------|
| | USP SIZE | LENGTH | |
| N 3347 | 1 | 100 cm | 1/2 Circle Round Bodied 40 mm Heavy |
| N 3348 | 1 (Loop) | 150 cm | 1/2 Circle Round Bodied 50 mm Heavy |
| S 3388 | 3-0 | 70 cm | Straight Cutting 60 mm |
| S 3389 | 0 | 70 cm | Straight Cutting 60 mm |
| S 3390 | 2-0 | 70 cm | Straight Cutting 60 mm |
| S 3437 | 3-0 | 75 cm | 1/2 Circle Round Bodied 20 mm |
| S3318S | 4-0 | 35 cm | 3/8 Circle Cutting 16 mm |
| S 903 | 2-0 | 2 X 76 cm | Without Needle |
| S 904 | 0 | 2 X 76 cm | Without Needle |
| S 905 | 1 | 2 X 76 cm | Without Needle |
| S 906 | 2 | 2 X 76 cm | Without Needle |
| 3L45CS75DA | 9-0 | 45 cm | 3/8 Circle Spatula DA |
| 3D150CZ40LP | 0 (Loop) | 150 cm | 3/8 Circle Round Bodied |
| 3C90DY40 | 1 | 90 cm | 1/2 Circle Cutting |



AXA PARENTERALS LTD.

AXALene

Monofilament Polypropylene
Non Absorbable Surgical Suture U.S.P.



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

AXALene

Monofilament Polypropylene
Non Absorbable Surgical Suture U.S.P.



AXALene – Monofilament Polypropylene Suture is a non absorbable sterile surgical suture composed of an isotactic crystalline stereoisomer of polypropylene, a synthetic linear polyolefin. The suture is pigmented blue to enhance visibility.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications:

Monofilament Polypropylene Suture is indicated for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological procedures.

- Smooth Surface results minimal tissue trauma
- Being a monofilament with smooth surface never support bacterial growth followed by capillary action
- Uniform elasticity supports small movements results cosmetic effect
- Ultra smooth surface results negligible pain
- Blue pigmentation enhances visibility

Best Used By:

- General Surgeons,
- Gynaecologists,
- Orthopaedics,
- Cardiac Surgeons,
- Endoscopic Surgeons

Ultimate Suture For:

- Skin Closure
- Hard Tissues approximation
- Ligament Repair
- Mesh Fixation

Additional Information:

| | | |
|-------------------------|---|--|
| Suture Characteristics | : | Non Absorbable Surgical Suture |
| Type | : | Monofilament |
| Material | : | Polypropylene |
| Color | : | Blue |
| Absorption | : | Non Absorbable |
| USP Range | : | 6/0 – 1 |
| Shelf Life | : | 5 Years |
| Sterilization | : | EO (Ethylene Oxide) |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction. |
| Packaging | : | In food grade folder packed in a medical grade pouch, with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXALene

Monofilament Polypropylene
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------|----------|--------|--|
| | USP SIZE | LENGTH | |
| AS 018 | 3-0 | 70 cm | 3/8 Circle Cutting 22mm |
| AS 018RC | 3-0 | 70 cm | 3/8 Circle Reverse Cutting 22mm |
| AS 805 | 0 | 70 cm | 1/2 Circle Taper Cutting 25mm |
| AS 806 | 2-0 | 90 cm | 1/2 Circle Taper Cutting 25mm |
| AS 807 | 2-0 | 70 cm | 1/2 Circle Taper Cutting 25mm |
| AS 822 | 3-0 | 70 cm | 1/2 Circle Round Bodied 25mm Double Armed |
| AS 822L | 3-0 | 90 cm | 1/2 Circle Round Bodied 26mm Double Armed |
| AS 823 | 6-0 | 70 cm | 3/8 Circle Slim Blade Cutting 15mm |
| AS 825 | 3-0 | 90 cm | 1/2 Circle Round Bodied 25mm |
| AS 829 | 6-0 | 70 cm | 3/8 Circle Round Bodied 13mm Double Armed |
| AS 829ASDA | 6-0 | 90 cm | 1/2 Circle Round Bodied 26mm Double Armed |
| AS 830 | 0 | 70 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 835 | 4-0 | 70 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 838 | 3-0 | 70 cm | 1/2 Circle Round Bodied 25mm |
| AS 839 | 3-0 | 70 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 840 | 1 | 70 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 840A | 1 | 100 cm | 1/2 Circle Round Bodied 30mm Heavy |
| AS 841 | 2-0 | 70 cm | 1/2 Circle Round Bodied 30mm |
| 8E75DZ30 | 2-0 | 75 cm | 1/2 Circle Round Body 30mm |
| 8K60CZ18DA | 8-0 | 60 cm | 3/8 Circle Reverse Cutting 18mm Double Armed |
| 8J75CZ10DA | 7-0 | 75 cm | 3/8 Circle Reverse Cutting 10mm Double Armed |
| 8I75CZ10DA | 6-0 | 75 cm | 3/8 Circle Reverse Cutting 10mm Double Armed |
| 8H75DZ16DA | 5-0 | 75 cm | 1/2 Circle Round Body 16mm Double Armed |
| 8H75DY16DA | 5-0 | 75 cm | 1/2 Circle Cutting 16mm Double Armed |
| 8H75CZ12 | 5-0 | 75 cm | 3/8 Circle Round Body 12mm |
| 8G150DZ17DA | 4-0 | 150 cm | 1/2 Circle Round Body 17mm Double Armed |
| 8G150CX19 | 4-0 | 150 cm | 3/8 Circle Reverse Cutting 19mm |
| 8F75DZ24 | 3-0 | 75 cm | 1/2 Circle Round Body 24mm |
| 8F75DZ18DA | 3-0 | 75 cm | 1/2 Circle Round Body 18mm Double Armed |

AXALene

Monofilament Polypropylene
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|-------------|-----------|--------|--|
| | USP SIZE | LENGTH | |
| AS 878 | 6-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 879 | 4-0 | 45 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 880 | 5-0 | 70 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 881 | 5-0 | 70 cm | 3/8 Circle Round Bodied 16mm |
| AS 882 | 5-0 | 70 cm | 3/8 Circle Round Bodied 16mm Double Armed |
| AS 882ASDA | 5-0 | 90 cm | 1/2 Circle Round Bodied 26mm Double Armed |
| AS 882CY | 5-0 | 70 cm | 3/8 Circle Cutting 16mm Double Armed |
| AS 883 | 1 | 100 cm | 1/2 Circle Reverse Cutting 45mm Heavy |
| AS 883/RB | 1 | 100 cm | 1/2 Circle Round Bodied 45mm Heavy |
| AS 887 | 3-0 | 70 cm | 1/2 Circle Round Bodied 20mm |
| AS 888 | 3-0 | 70 cm | Straight Cutting 60mm |
| AS 894 | 0 | 90 cm | 1/2 Circle Taper Cutting 30mm Heavy |
| AS 896 | 1 | 100 cm | 1/2 Circle Taper Cutting 30mm Heavy |
| AS 898 | 2-0 | 70 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 878VS | 6-0 | 70 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 880VS | 5-0 | 90 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 883MB | 1 | 45 cm | 1/2 Circle Reverse Cutting 40mm Heavy |
| AS 857 | 4-0 | 90 cm | 1/2 Circle Taper Cutting 17mm |
| 8DA8F75CY24 | 3-0 | 75 cm | 3/8 Circle Cutting 24mm |
| 8E75CY26 | 2-0 | 75 cm | 3/8 Circle Cutting 26mm |
| 8E75CY20 | 2-0 | 75 cm | 3/8 Circle Cutting 20mm |
| 8D150DZ40LP | 0 (Loop) | 150 cm | 1/2 Circle Round Body 40mm |
| 8D100DZ40 | 0 | 100 cm | 1/2 Circle Cutting 40mm |
| 8D100DY40 | 0 | 100 cm | 1/2 Circle Cutting 40mm |
| 8C75CZ36 | 1 | 75 cm | 3/8 Circle Round Body 36mm |
| 8C150DZ40LP | 1 (Loop) | 150 cm | 1/2 Circle Round Body 40mm |
| 8C150DZ40 | 1 | 150 cm | 1/2 Circle Round Body 40mm |
| 8B76DY50 | 2 | 76 cm | 1/2 Circle Cutting 50mm |
| 8C100DZ40 | 1 | 100 cm | 1/2 Circle Round Body 40mm |



AXA PARENTERALS LTD.

AXASIL

Black Braided Silk
Non-Absorbable Suture (U.S.P.)



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

AXASIL

Black Braided Silk

Non Absorbable Surgical Suture U.S.P.



AXASIL– Black Braided Silk Suture is a non absorbable, sterile, surgical suture composed of an organic protein called fibroin. This protein is derived from the domesticated species *Bombyx mori* (B. Mori) of the family Bombycidae.

AXASIL sutures are processed to remove the natural waxes and gums. AXASIL suture is dyed black and coated with a special wax mixture. AXASIL Black Braided Silk is also available in its natural color.

AXASIL Virgin silk is available in which the sericin gum is not removed and serves to hold the filaments together. In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications:

Black Braided Silk Suture is indicated for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological procedures.

- Highest purified protein from Species B. Mori results no tissue reaction
- Beeswax coated ultra smooth surface decrease suturing timing followed by no tissue trauma
- Unmatched Knot strength provides satisfaction to user
- USP approved dye increases visibility followed by no reaction

Best Used By:

- Orthopaedic Surgeons,
- General Surgeons,
- Gynaecologists,
- Cardiac Surgeons,
- Endoscopic Surgeons

Ultimate Suture For:

- Skin Closure
- Hard Tissues approximation
- Tendon Repair
- Muscles approximation

Additional Information:

| | | |
|-------------------------|---|--|
| Suture Characteristics | : | Non Absorbable Surgical Suture |
| Type | : | Braided |
| Material | : | Braided Silk |
| Color | : | Black |
| Absorption | : | Non Absorbable |
| USP Range | : | 5/0 – 2 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction. |
| Packaging | : | In food grade folder packed in a medical grade pouch, with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXASIL

Black Braided Silk
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|------------|----------|--------|---------------------------------|
| | USP SIZE | LENGTH | |
| AS 5001 | 4-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 5002 | 3-0 | 76 cm | 3/8 Circle Cutting 16mm |
| AS 5003 | 2-0 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 5003RC | 3-0 | 75 cm | 3/8 Circle Reverse Cutting 16mm |
| AS 5004 | 0 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 5005 | 1 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 5006 | 2 | 75 cm | 1/2 Circle Reverse Cutting 38mm |
| AS 5017 | 0 | 76 cm | 3/8 Circle Round Bodied 30mm |
| AS 5028 | 3-0 | 76 cm | 3/8 Circle Reverse Cutting 26mm |
| AS 5026 | 5-0 | 76 cm | 3/8 Circle Reverse Cutting 8mm |
| AS 5029 | 6-0 | 76 cm | 3/8 Circle Round Bodied 13mm |
| AS 5036 | 2-0 | 76 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 5036A | 2-0 | 35 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 5037 | 0 | 76 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 5037A | 0 | 35 cm | 3/8 Circle Reverse Cutting 45mm |
| AS 5049 | 4-0 | 76 cm | 1/2 Circle Round Bodied 16mm |
| AS 5052 | 2-0 | 76 cm | 1/2 Circle Reverse Cutting 50mm |
| AS 5061PCM | 4 | 76 cm | 1/2 Circle Round Bodied 35mm |
| AS 5062 | 1 | 76 cm | 3/8 Circle Cutting 60mm |
| AS 5070 | 3-0 | 76 cm | 1/2 Circle Round Bodied 25mm |
| AS 5080 | 5-0 | 76 cm | 3/8 Circle Reverse Cutting 12mm |
| AS 5082 | 4-0 | 76 cm | 3/8 Circle Round Bodied 16mm |
| AS 5083 | 3-0 | 76 cm | 1/2 Circle Round Bodied 35mm |
| AS 5086 | 4-0 | 76 cm | 1/2 Circle Round Bodied 20mm |
| AS 5087 | 3-0 | 76 cm | 1/2 Circle Round Bodied 20mm |
| AS 5090 | 5-0 | 76 cm | Straight Round Bodied 20mm |
| AS 5096 | 5-0 | 76 cm | 3/8 Circle Round Bodied 20mm |
| AS 5098 | 5-0 | 76 cm | 1/2 Circle Taper Cutting 17mm |
| AS 5290 | 2-0 | 76 cm | Straight Cutting 60mm |
| AS 5333 | 2-0 | 76 cm | 1/2 Circle Round Bodied 30mm |

AXASIL

Black Braided Silk
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|--------------------------------------|----------|-----------|------------------------------------|
| | USP SIZE | LENGTH | |
| N 5333DY | 2-0 | 76 CM | 1/2 Circle Cutting 30mm |
| N 5334 | 0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| N 5610 | 2-0 | 76 cm | 1/2 Circle Taper Cutting 17mm |
| N 5670 | 2-0 | 76 cm | 1/2 Circle Taper Cutting 25mm |
| N 5001VS | 4-0 | 90 cm | 3/8 Circle Cutting 16mm |
| N 5002VS | 3-0 | 90 cm | 3/8 Circle Cutting 16mm |
| N 5085 | 3-0 | 90 cm | 1/2 Circle Round Bodied 25mm |
| N 5332N | 3-0 | 76 cm | 1/2 Circle Round Bodied 30mm |
| N 5335 | 0 | 75 cm | 3/8 Circle Round Bodied 25mm |
| N 5011 | 1 | 110 cm | 1/2 Circle Taper Cutting 30mm |
| N 5063 | 1 | 76 cm | 1/2 Circle Round Bodied 40mm Heavy |
| S 212 | 3-0 | 2 x 75 cm | Without Needle |
| S 213 | 2-0 | 2 x 75 cm | Without Needle |
| S 214 | 0 | 2 x 75 cm | Without Needle |
| S 215 | 1 | 2 x 75 cm | Without Needle |
| S 216 | 2 | 2 x 75 cm | Without Needle |
| S 217 | 3 | 2 x 75 cm | Without Needle |
| Black Braided Silk Non Sterile Reels | | | |
| R 826 | 2 | 25 m | 6 Reels X 25 Meters |
| R 825 | 1 | 25 m | 6 Reels X 25 Meters |
| R 824 | 0 | 25 m | 6 Reels X 25 Meters |
| R 823 | 2-0 | 25 m | 6 Reels X 25 Meters |
| R 822 | 3-0 | 25 m | 6 Reels X 25 Meters |
| R 821 | 4-0 | 25 m | 6 Reels X 25 Meters |
| 5E75DZ30 | 2-0 | 75 cm | 1/2 Circle Round Bodied 30mm |
| 5G75CY16 | 4-0 | 75 cm | 3/8 Circle Cutting 16mm |
| 5H75CY16 | 5-0 | 75 cm | 3/8 Circle Cutting |
| 5F75CZ26 | 3-0 | 75 cm | 3/8 Round Bodied |
| 5F75CY25 | 3-0 | 75 cm | 3/8 Circle Cutting |
| 5F13X60 | 3-0 | 13x60 | Without Needle |

AXASIL

Black Braided Silk
Non Absorbable Surgical Suture U.S.P.



| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|----------|----------|--------|------------------------------|
| | USP SIZE | LENGTH | |
| 5E75CY26 | 2-0 | 75 cm | 3/8 Circle Cutting |
| 5E75CX24 | 2-0 | 75 cm | 3/8 Circle Reverse Cutting |
| 5E12X60 | 2-0 | 12x60 | Without Needle |
| 5D75CY40 | 0 | 75 cm | 3/8 Circle Cutting |
| 5C75CY40 | 1 | 75 cm | 3/8 Circle Cutting |
| 5E13X60 | 2-0 | 13x60 | Without Needle |
| 5G75CZ16 | 4-0 | 75 cm | 3/8 Circle Round Bodied 16mm |
| 5F75DZ30 | 3-0 | 75 cm | 1/2 Circle Round Bodied 30mm |



AXA PARENTERALS LTD.

AXABond

Braided & Coated Polyester
Non Absorbable Surgical Suture U.S.P.



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com

AXABond

Braided & Coated Polyester
Non Absorbable Surgical Suture U.S.P.



AXABond – Braided & Coated Polyester Suture is a non absorbable, braided, sterile, surgical suture composed of Poly (ethylene terephthalate). It is prepared from fibres of high molecular weight, long-chain, linear polyesters having recurrent aromatic rings as an integral component. Braided Polyester Sutures are braided for optimal handling properties and for good visibility in the surgical field.

In-house needle manufacturing provides you the better needles with optimum sharpness which finally delivers smooth riding on tissues during suturing.

Indications:

AXABond - Braided & Coated Polyester Suture is indicated for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological procedures.

- High Molecular weight , Long Chain, Linear Polyester's fibers results in ultra strong suture
- Smooth Surface results easy passage through toughest tissues
- Ultra high strength offers extra support required by tissues
- AXABond – a choice of Leading Orthopaedic Surgeons

Best Used By:

- Orthopaedics
- Cardiac Surgeon
- General Surgeon

Additional Information:

| | | |
|-------------------------|---|--|
| Suture Characteristics | : | Non Absorbable Surgical Suture |
| Type | : | Braided |
| Material | : | Polyester |
| Color | : | Green / White |
| Absorption | : | Non Absorbable |
| USP Range | : | 5/0 – 5 |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Excellent tensile strength, Excellent knot tying, Minimum tissue reaction. |
| Packaging | : | In food grade folder packed in a medical grade pouch, with or without needle |
| Needle Type | : | Round bodied, cutting edge, taper point, straight, blunt point |
| MFG. LIC No | : | MFG/MD/2020/000109 |
| OEM. LIC No | : | N. Code: KA/DEVICE/MFG/MD/2020/000109 |

AXABond

Braided & Coated Polyester
Non Absorbable Surgical Suture U.S.P.




| Code No. | SUTURE | | NEEDLE DESCRIPTION |
|---------------|-----------|----------|--|
| | USP SIZE | LENGTH | |
| AS 617 | Green 2-0 | 90 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 617A | White 2-0 | 90 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 625 | Green 3-0 | 90 cm | 1/2 Circle Round Bodied 25mm Double Armed |
| AS 632 | Green 3-0 | 76 cm | 1/2 Circle Round Bodied 25mm |
| AS 635 | Green 4-0 | 76 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 636 | Green 3-0 | 76 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 637 | Green 2-0 | 76 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 637A | White 2-0 | 76 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 640 | Green 4-0 | 45 cm | Sraight Cutting 53mm Double Armed |
| AS 643 | Green .2 | 100 cm | 1/2 Circle Taper Cutting 45mm Heavy |
| AS 645 | Green 5 | 100 cm | 1/2 Circle Trocar Point 55mm Heavy |
| AS 652 | Green 3-0 | 76 cm | 1/2 Circle Round Bodied 25mm Double Armed |
| AS 669 | Green 3-0 | 90 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 672 | Green 4-0 | 76 cm | 3/8 Circle Round Bodied 16mm |
| AS 677 | Green 2-0 | 76 cm | 1/2 Circle Taper Cutting 25mm Double Armed |
| AS 677A | White 2-0 | 76 cm | 1/2 Circle Taper Cutting 25mm Double Armed |
| AS 679 | Green 0 | 100 cm | 1/2 Circle Reverse Cutting 50mm |
| AS 680 | Green1 | 100 cm | 1/2 Circle Reverse Cutting 50mm |
| AS 687 | Green 2-0 | 90 cm | 1/2 Circle Taper Cutting 25mm Double Armed |
| AS 687A | White 2-0 | 90 cm | 1/2 Circle Taper Cutting 25mm Double Armed |
| AS 690 | Green 5-0 | 76 cm | 3/8 Circle Round Bodied 12mm Double Armed |
| AS 691 | Green 4-0 | 90 cm | 3/8 Circle Round Bodied 12mm Double Armed |
| AS 695 | Green 4-0 | 90 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| AS 696 | Green 5-0 | 76 cm | 3/8 Circle Round Bodied 20mm Double Armed |
| AS 687L | White 2-0 | 75 cm | 1/2 Circle Taper Cutting 26mm Double Armed |
| 6E90DZ26DA | 2-0 | 90 cm | 1/2 Circle Round Body 26mm Double Armed |
| 6E75DZ20DAPL | 2-0 | 75 cm | 1/2 Circle Round Body 20mm Double Armed |
| 6E75DZ20DA | 2-0 | 75 cm | 1/2 Circle Round Body 20mm Double Armed |
| 6E75DK18DA | 2-0 | 75 cm | 1/2 Circle Taper Cutting 18mm Double Armed |
| 6E75DK17DAPL | 2-0 | 75 cm | 1/2 Circle Taper Cutting 17mm Double Armed |
| 6E10x75DK26DA | 2-0 | 10X75 cm | 1/2 Circle Taper Cutting 26mm Double Armed |



AXA PARENTERALS LTD.


AXAMesh

Monofilament Polypropylene (Undyed)
Non Absorbable Sterile Surgical Mesh




AXAMesh
Monofilament Polypropylene (Undyed)
Non Absorbable Sterile Surgical Mesh


For Hernia Repair



Manufactured in India by:
UNISUR LIFECARE PVT. LTD.
#15/1,2,3, Andrahalli Main Road,
Acharya Indl. Complex,
Vishwanandam Pk., Bangalore - 560091
CIN: U33110KA2015PTC079811
Mfg. Lic. No.: KTK/28/415/2013


CE 2265
EN ISO 13485 Certified
STERILE EO

Marketed by:
 **AXA PARENTERALS LTD.**
Axa House, Kishanpur,
Jabalpur, Roorkee-247667
Uttarakhand, India
www.axaparenterals.com



AXAMesh

Monofilament Polypropylene (Undyed)
Non Absorbable Sterile Surgical Mesh

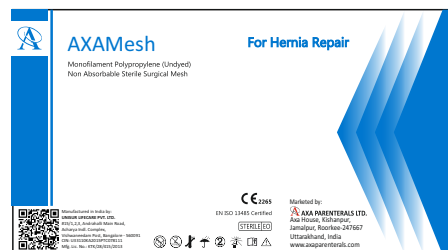

AXAMesh – Monofilament Polypropylene Mesh (undyed)

Polypropylene Mesh is constructed of knitted filaments of extruded polypropylene identical in composition to that used in AXAlene Polypropylene Suture. This material, when used as a suture, has been reported to be nonreactive and to retain its strength indefinitely in clinical use.

Polypropylene Mesh is knitted by a process which interlinks each fiber junction and which provides for elasticity in both directions. This construction permits the mesh to be cut into any desired shape or size without unraveling. This bi-directional elastic property allows adaptation to various stresses encountered in the body

In-house mesh manufacturing makes us unique and potential manufacturer. AXAMesh is our unique product and we are available with any type, any kind of modification.

Indications of Polypropylene Mesh:

Polypropylene Mesh used for the hernia repair and other facial deficiencies that require the addition of a reinforcing or bridging material to obtain the desired surgical result.

A Hernia Mesh is a Non-Absorbable surgical mesh, used to span and reinforce traumatic or surgical wounds to provide extended support during and following wound healing. The material is neither absorbed nor is it subject to degradation or weakening by the action of tissue enzymes.

- AXAMesh Probably the best in kind
- An artificial wall which doesn't feel
- Latest technology with perfection provides ultra smooth surface results optimum healing
- Knitting is as per requirement of tissues results no discomfort after surgery.

Best used and ultimate mesh for:


Polypropylene Mesh best used by General Surgeon, Gynaecologist, Endoscopic Surgeons and its an Ultimate Mesh for Hernia Repair.

Additional Information:

| | | |
|-------------------------|---|--|
| Suture Characteristics | : | Non Absorbable Surgical Suture |
| Type | : | Knitted filaments of extruded polypropylene |
| Material | : | Polypropylene |
| Color | : | White (undyed) |
| Absorption | : | Non Absorbable |
| Sterilization | : | EO (Ethylene Oxide) |
| Shelf Life | : | 5 Years |
| Product Characteristics | : | Light weight construction because of which less foreign mass in the body & flexible scar tissue. Macro porous for high visibility and colonization |
| MFG. LIC No | : | KTK/28/441/2016 |
| OEM. LIC No | : | N. Code: KR/DRUGS/KTK/28/441/2016 |

AXAMesh

Monofilament Polypropylene (Undyed)
Non Absorbable Sterile Surgical Mesh

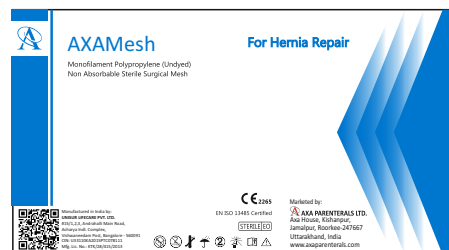

| Code No. | Mesh Size | No. of Units / Box |
|----------|-------------|--------------------|
| AS 0306 | 3cm X 6cm | 6 units / box |
| AS 0510 | 5cm X 10cm | 6 units / box |
| AS 0611 | 6cm X 11cm | 6 units / box |
| AS 0815 | 8cm X 15cm | 6 units / box |
| AS 1015 | 10cm X 15cm | 3 units / box |
| AS 1215 | 12cm X 15cm | 3 units / box |
| AS 1515 | 15cm X 15cm | 3 units / box |
| AS 1530 | 15cm X 30cm | 1 unit / box |
| AS 3030 | 30cm X 30cm | 1 unit / box |



AXA PARENTERALS LTD.



AXA PARENTERALS LTD.

Plot No. 936, 937 & 939, Vill- Kishanpur, Jamalpur, Roorkee - 247667,
Distt. Haridwar, Uttarakhand, INDIA. www.axapar.com



axaparr



axapar



LtdAxa



Axa Parenterals Ltd



Axapar.com